

TLC Packer

User guide

Additional details at our "Instructions" page

TLC Tugger

Proudly helping men restore
www.TLCTugger.com

Ron Low

President 847 414-1692

TLCTugger@Juno.com



The TLC Packer8



Step 1: (This user has enough slack to use the P16 effectively.) Apply lotion to the vertical sidewall of the Packer.



Step 2: Roll the skin back and place the recessed base of the Packer against the glans.



Step 3: Nudge the Packer and glans back with one hand while rolling skin onto the Packer with the other.



Step 4: Pull the skin outwardly past the Packer tip and even it out until you can bunch up a ring of skin.



Step 5: Pop the Your-Skin Cone on and check for uniformity.



Step 6: Wear the device under loose briefs or nude in bed.

Step 7: Hygiene

- a) Wash the devices daily
- b) Remove the devices to urinate
- c) An alternate version of the Packer with a tugging handle is called the TLC-Stretch or VLC-Stretch. The handle might provide additional useful leverage for installing the device, and it gives a means of attaching a failsafe lace for wearing under shorts/without briefs.
- d) The recommended lotion is called Eucerin. It comes in 16 ounce bottles.